



# Preparing for Conversations With Floodlight™ MS






Floodlight MS may provide insight into neurologic function to aid you and your patients to better understand their MS. The Floodlight MS portal is your gateway to your patient’s objective data, which may provide insights into what may be happening with your patient’s physical and cognitive function between visits. You also have the ability to add members of your care team within the portal to help you effectively care for your patients.

## TO HELP PREPARE FOR THE FOLLOW-UP APPOINTMENT WITH YOUR PATIENT

1. Identify that your patient is using Floodlight MS.
2. During your patient’s pre-appointment reminder, consider reminding them to complete their Patient Pre-Visit Checklist and to come ready to discuss their data.
3. Log in to the Floodlight MS portal and pull up your patient’s Floodlight MS profile by clicking “My Patients” and entering their birthdate, first name, or last name into the search bar and then clicking on their name.
4. Review your patient’s Floodlight MS data over a 3-, 6-, or 9-month period.
5. Generate PDF reports, or export a patient’s data as a spreadsheet to share with your team or integrate with any third-party tools you may use.
6. Change your patient’s data visualization settings by checking or unchecking the “Trend Line” box in your patient’s data dashboard. For more information on the Floodlight MS graphs and for more insight into your patient’s activities, click on the triangle in the upper right corner of the graph.
7. Consider reviewing your patient’s Floodlight MS data together with your patient’s medical record. Based on any patterns you observe from the Floodlight MS data, consider:
  - Trends that may indicate changes in performance
  - Further evaluation of a specific domain to review in conjunction with the Floodlight MS data

## FLOODLIGHT MS ACTIVITIES ARE INTENDED TO PROVIDE ADDITIONAL DATA FOR YOUR PRACTICE

DOMAINS	 HAND FUNCTION		 WALKING ABILITY (GAIT AND DYNAMIC BALANCE)		 COGNITIVE FUNCTION
FLOODLIGHT MS ACTIVITIES	Pinch a Tomato	Draw a Shape	U-Turn	2-Minute Walk	Match the Symbols
DEMONSTRATED CONCORDANCE WITH WHICH IN-CLINIC ASSESSMENT?	Nine-hole peg test (9HPT)		5 U-Turn tests and timed 25-foot walk (T25FW)		Symbol Digit Modalities Test (SDMT)

Source: Midaglia L, et al. *J Med Internet Res*. 2019;21:e14863.

Please see the user manuals on FloodlightMS.com for directions on how to properly perform the activities in the app.

**These activities only provide objective measurements of function and do not provide an interpretation or a clinical implication of the measurements.**

EMR, electronic medical records; MS, multiple sclerosis.



# Conducting the Floodlight MS Follow-up Appointment

Your patient has been busy completing their Floodlight MS activities and Journal entries since your last visit together. Now is the time to review both their Floodlight MS objective data and patient-reported outcomes that may help with improving clinical conversations about better understanding their MS.

## DURING THE FOLLOW-UP APPOINTMENT, CONSIDER

1. **Showing** your patient their Floodlight MS data and explain any of your findings.
2. **Discussing** your patient's experience using Floodlight MS and reviewing their Patient Pre-Visit Checklist. See below for some sample questions and conversation starters.
3. **Evaluating and adjusting** your patient's personalized Floodlight MS routine.
4. **Reminding** your patient about the importance of continuing to use Floodlight MS regularly to help better facilitate conversations during the next and following visits.
5. **Setting** your patient's next appointment and having them record it in the app.
6. **Reminding** your patient to complete their post-visit questionnaire in the app.

### FLOODLIGHT MS PATIENT CONVERSATION STARTERS

- How easy was it for you to complete your Floodlight MS activities regularly? Were any activities more difficult than others?
- Did anything get in the way of you completing your Floodlight MS activities (for example: life events, illness, vacation, etc)?
- Is there anything you want to talk about based on your Floodlight MS data?



### FLOODLIGHT MS HANDLES DATA SECURELY

Floodlight MS is designed in compliance with applicable privacy laws, such as HIPAA.



### GETTING SUPPORT

Floodlight MS customer support is available for any questions related to your Floodlight MS experience. Reach support by calling **1-888-ILLUMN8 (1-888-455-8668)** or emailing **[FloodlightMS\\_support.us@gene.com](mailto:FloodlightMS_support.us@gene.com)**.

HIPAA, Health Insurance Portability and Accountability Act.

**Genentech**  
A Member of the Roche Group

© 2021 Genentech USA, Inc. All rights reserved. M-US-00007850(v3.0) 07/21